Oral-B

TEST DRIVE

POWER BRUSH TRIAL PROGRAM

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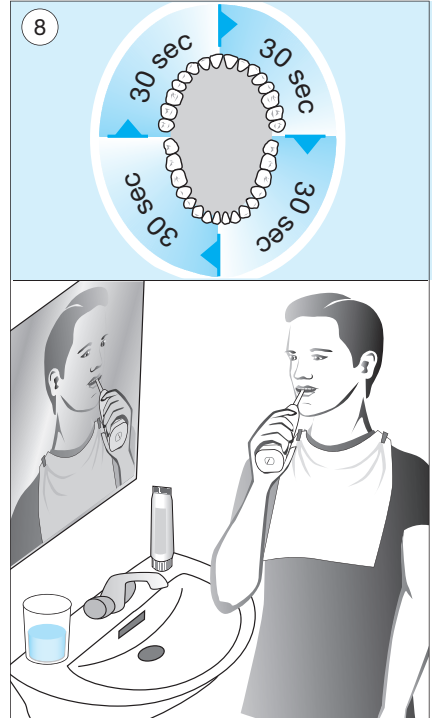
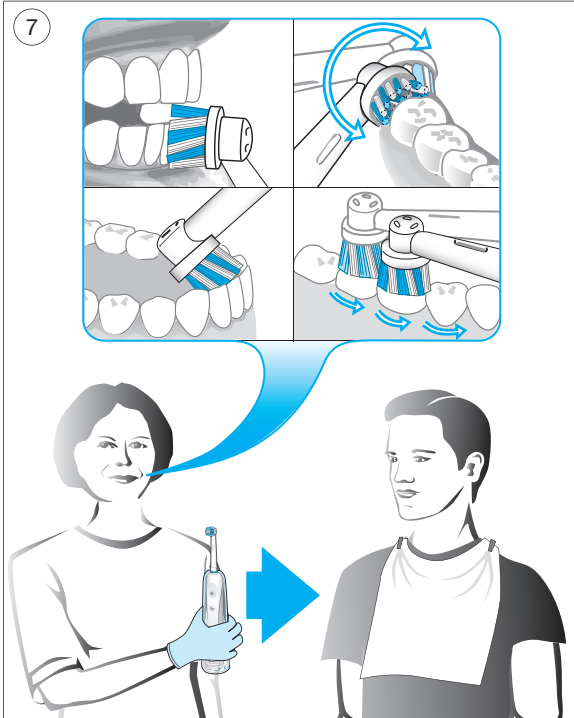
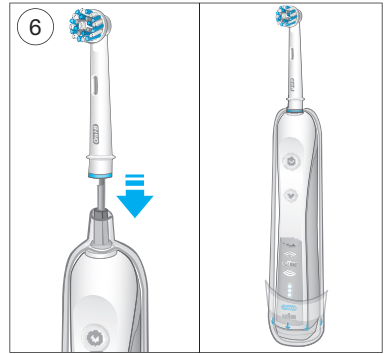
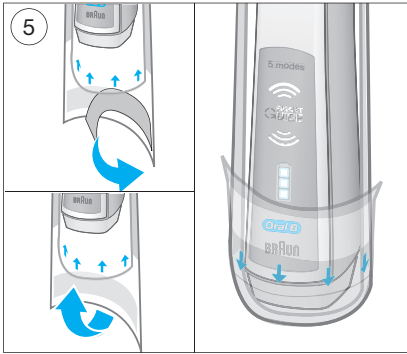
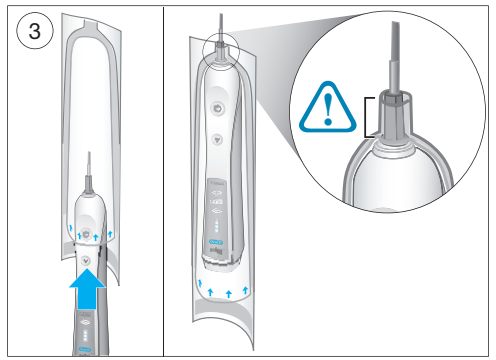
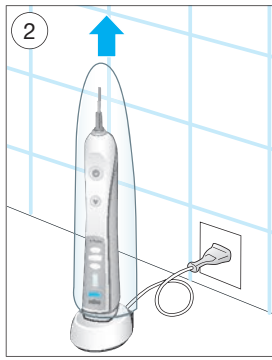
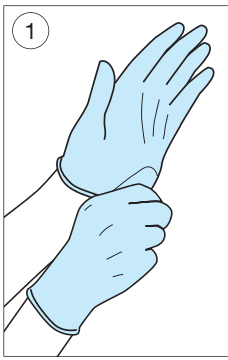
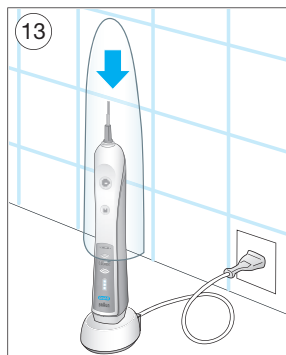
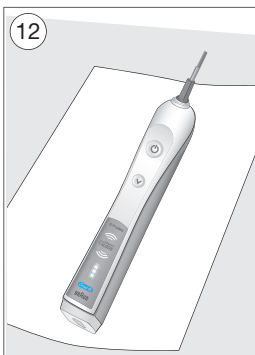
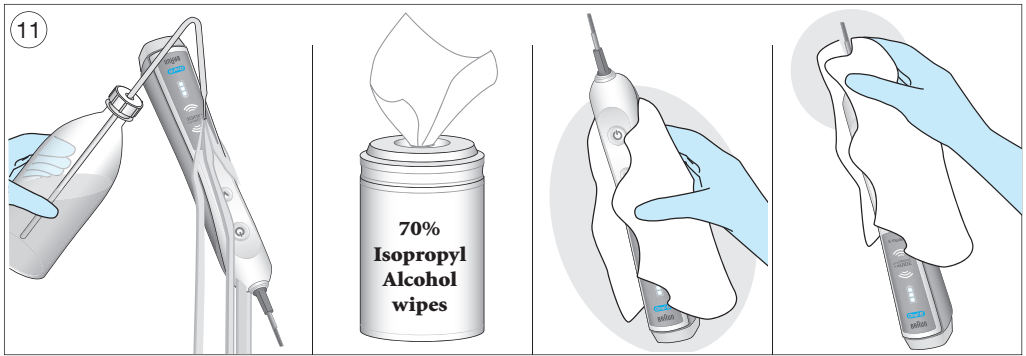
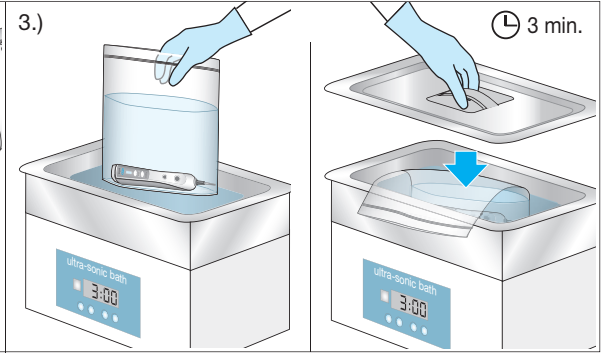
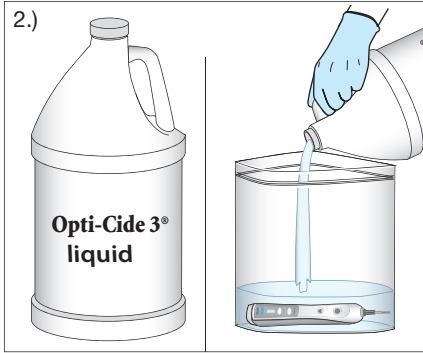
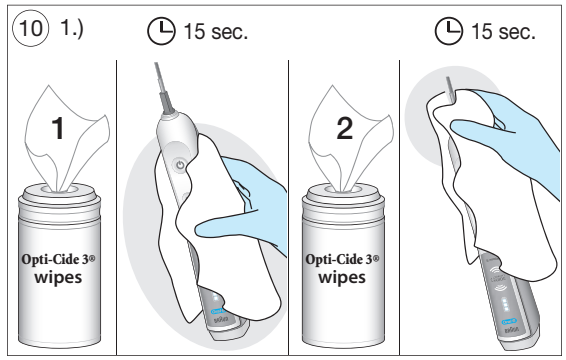
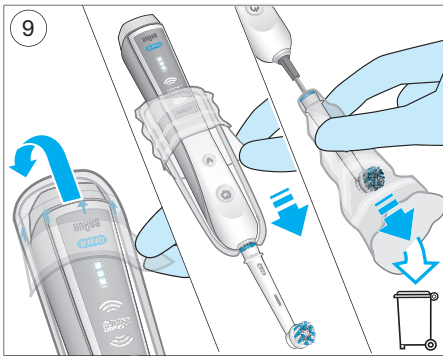
English

Disinfection and Handling Manual for Dental Professionals – for shared-handle Power Brush Trials.

Before using this toothbrush, please read the following disinfection and handling instructions and save this manual for future reference. Please also refer to the separate usage instruction manual for the toothbrush handle.

Indications for Use: The Oral-B® Test Drive Power Brush Trial Program Kit is intended for use as a power toothbrush to promote good oral hygiene, including the reduction of dental plaque and the treatment and prevention of gingivitis.

The Oral-B® Test Drive Power Brush Trial Program Kit is indicated for use under the supervision of a dental professional as part of the Oral-B® Test Drive Power Brush Trial Program.

- ① Put on gloves.
- ② Take the brush handle off the charger and remove the protective cover.
- ③ Hold the brush handle with the buttons facing you and carefully slide it into one of the protective sheaths from the kit, until the metal shaft extends through the small opening on the sheath. Ensure the opening of the sheath is even with the top of the orange rod.
- ④ Remove the back paper off the sheath.
- ⑤ Close the sheath at the bottom by peeling away the tape to expose adhesive, then fold it across the bottom of the handle and seal the sheath closed.
If the sheath is damaged in any way, discard it and place a new sheath on the handle being careful not to damage or pierce the sheath.
- ⑥ Take a new brush head and place it onto the handle. The brush head will fit over the top of the sheath. Slide the brush head all the way down onto the handle until it fits tightly. Verify that there is no gap between the brush head and the collar of the handle.
- ⑦ Before use, wet the brush head with water. Provide brushing instructions to the patient and allow them to brush.
Note: Please refer to the separate usage instructions for the toothbrush.
- ⑧ Recommend to the patient to brush over a sink in front of a mirror. Allow 2 full minutes for the trial.
Use proper infection control techniques when

handling the toothbrush after it has been used by the patient.

- ⑨ Carefully open the sheath at the bottom. Hold the handle upside-down to prevent saliva from running down the clean handle. Peel the sheath down over the brush head and remove both the sheath and brush head by pulling them off together. Dispose of both sheath and brush head properly.
- ⑩ To clean and disinfect the handle after each patient trial, use Opti-Cide 3® * Disinfection Wipes and Liquid Disinfection Solution.
Note: Do not put in autoclave
 1. Cleaning and Disinfection Step 1: Wipe
Using a disinfection wipe, wrap the wipe around the handle and wipe the handle and shaft up and down for 15 seconds, ensuring that all surfaces are cleaned. Repeat, using a 2nd clean disinfection wipe for 15 seconds, ensuring that all surfaces are cleaned.
 2. Cleaning and Disinfection Step 2: Immerse
Place the handle in a 1-gallon plastic zipper bag, so that it is laying flat (horizontal). Take care that the metal tip does not pierce the bag. Fill the bag with the disinfection solution so that the handle is fully covered with solution. Carefully seal the bag.
 3. Cleaning and Disinfection Step 3: Ultrasonic
Place the sealed zipper bag in the ultrasonic bath and turn it on for 3 minutes. Lay the top of the bag over the edge of the ultrasonic unit so that the top of the bag will not submerge. Put on the lid to hold the bag in place.

At the end of 3 minutes, remove the zipper bag from the ultrasonic bath. Open the bag and remove the handle.

Dispose of the disinfection solution and plastic zipper bag after each use.
- ⑪ Rinse the handle with sterile or de-ionized water. Wipe the handle with a 70% isopropyl alcohol wipe, to aid in drying the handle.
- ⑫ Lay the handle on a paper towel and allow it to air-dry.
- ⑬ Place the cleaned and disinfected handle in a protective cover and then place onto the charger somewhere dry and clean until the next patient trial.
Note: Store and use at room temperature (60–95 °F).
Note: A damaged or non-functioning unit/toothbrush should no longer be used. Do not

modify or repair the product. Contact Oral-B® representative for replacement.

* Opti-Cide 3® is a trademark of Micro-Scientific®

IMPORTANT – Please record in the table at the back of this manual each time the Test Drive handle has been cleaned and disinfected.

The Test Drive handle should not be used more than 240 times. Please contact your Oral-B® representative for a replacement.

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